



POWERLIFT 500 CLASSIC LIFT

Designed to meet the needs of the caregiver as well as the patient. Caregivers and patients like the Powerlift 500 Classic Lift because of the spread on the sling cradle. In simple terms, the hooks are spread out so they don't pinch the patient. This type of spreader allows use of a greater variety of slings.

The Powerlift 500 Classic Lift features ergonomic handle grips to reduce back strain and a mechanical down-release lever for non-powered lowering. The single-speed actuator is reverse-mounted to give the patient increased leg clearance, and the unit also features leg guards and caster bumpers to prevent damage to walls and furniture.

The Powerlift 500 Classic Lift has a low base that needs only 5.5" clearance. This makes it easy to roll under a bed or chair. The 6-point cradle insures you'll be able to use a variety of 4- and 6-point slings.

HAND CONTROL

- The hand control activates the lifter and has two buttons; one "up" and one "down".
- Simply press the button for the direction you want the lifter to move.

BATTERY CHARGING

- Plug the charger into the wall socket (110 V) prior to connecting the plug to the control box.
- If the unit sits idle for an extended period of time, the battery life diminishes. A control box with battery should be charged at least every six months. However, the longest life is obtained when the battery is fully charged.
- After approximately 6 hours of charging, the batteries will be fully charged.



Key Product Features

- Smooth, quiet transfer reduces patient anxiety
- Innovative 6-point non-sway cradle offers extra stability
- Cradle design works with Hoyer 4-point and 2-point slings
- Adjustable U-base
- Built-in emergency stop button
- Manual release for emergency lowering
- Ergonomic handle grips for easier positioning
- Improved patient legroom with reverse-mounted actuator
- Leg guards and caster bumpers prevent wall and furniture damage
- 400 lb. patient capacity