



# SVED® System Do's and Don'ts

1

**Do NOT** place foam on intact skin




2

**Do NOT** over-fill or tightly pack wound with foam. Allow the wound to collapse around foam for faster healing



3

Count and document foam placed into and removed from the wound at **each** dressing change

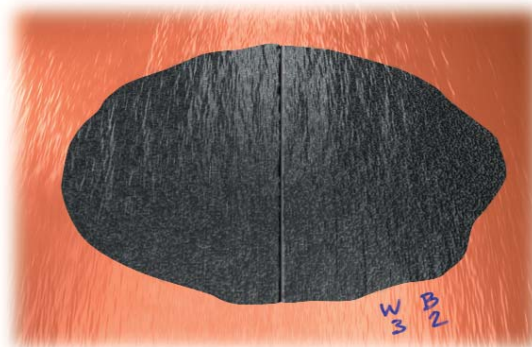

Innovative Therapies Inc.  
12 Meem Avenue, Suite C, Gaithersburg, MD 20877  
• tel: 866-ITI-NPWT (484-6798) • fax: 301-987-1133 • e-mail: customerservice@itinpwt.com  
[www.itinpwt.com](http://www.itinpwt.com)

SVED is a registered trademark of Innovative Therapies Inc.  
Innovative Therapies Inc. owns, uses, or has applied for the following trademark(s) or service mark(s): Svedman, SVED, SensiSkin, SpeedConnect, "Outcomes Focused Wound Therapy." All other trademarks are trademarks of their respective owners or holders. ©2011 Innovative Therapies Inc.  
Please refer to the ITI product manual for indications, contraindications, precautions and safety information. Always consult a physician and product instructions for use prior to application.  
Caution: Federal law restricts this device to sale by or on the order of a physician.